

Repair of Recurrent Rectal Prolapse with Perineal Approach and Utilization of a Next Generation Decellularized Dermal Allograft to Augment the Repair of the Pelvic Floor Hernia

Anna M. Toker, MD, FACS, FASCRS
Arlington, TX

Board Certified by the American College of Surgeons and the American Society of Colon and Rectal Surgeons

Surgical Summary

Clinical Presentation:

- 90-year-old female with significant medical history of diabetes and hypertension underwent a perineal repair and levatorplasty for rectal prolapse 2 years ago, presented to the office with complaints of recurrent prolapse, outlet obstruction constipation with incontinence, and pelvic pain.
- On exam she had a classic rocker bottom defect with decreased tone, significant perineal laxity, and thin watery mucous discharge. The anterior rectal wall was partially prolapsed at rest.

Intraoperative Findings:

- A recurrent rectal prolapse with incompetent rectovaginal septum and levator muscle diastasis consistent with pelvic floor dysfunction and laxity. (Figure 1)

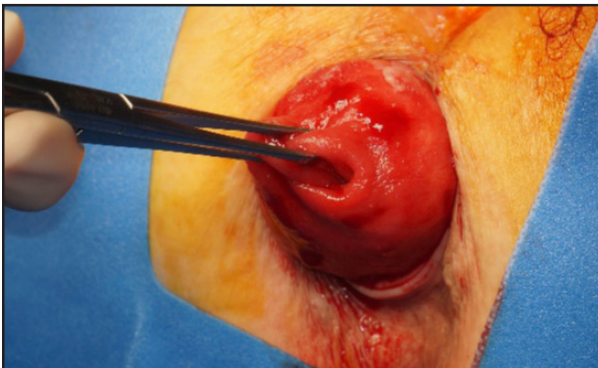


Figure 1. A recurrent rectal prolapse with incompetent rectovaginal septum and levator muscle diastasis consistent with pelvic floor dysfunction and laxity.

Surgeon Perspective:

“The use of this decellularized dermal allograft in this patient allowed for a more thorough repair of a pelvic floor hernia by providing supplemental support, without tension and without artificial materials.”

Repair of Recurrent Rectal Prolapse with Perineal Approach and Utilization of a Next Generation Decellularized Dermal Allograft to Augment the Repair of the Pelvic Floor Hernia.

Anna M. Toker, MD, FACS, FASCRS
Arlington, TX

Board Certified by the American College of Surgeons and the American Society of Colon and Rectal Surgeons

Surgical Procedure

- Resection of rectal prolapse via perineal approach. (Figure 2)
- Implanted DermaPure® decellularized dermal allograft in the rectovaginal septum (Figure 3) and along the levator muscle shelf. (Figure 4)
- Coloanal anastomosis performed. (Figure 5)

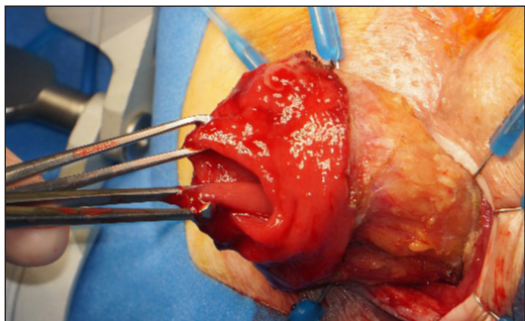


Figure 2. Resection of rectal prolapse via perineal approach.

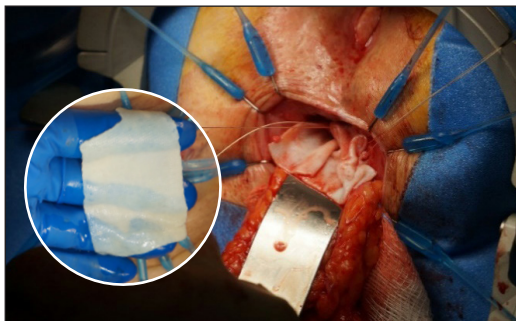


Figure 3. Implantation of DermaPure® in the rectovaginal septum.

**DermaPure®
Implantation:
7 cm x 10 cm**

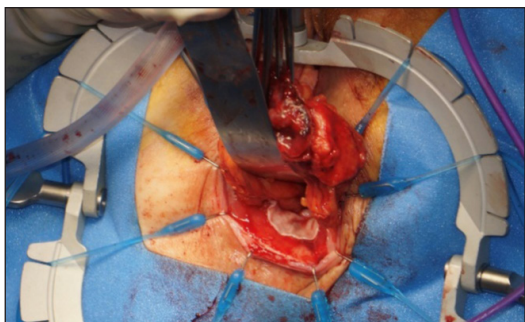


Figure 4. Implantation of DermaPure® along the levator muscle shelf.

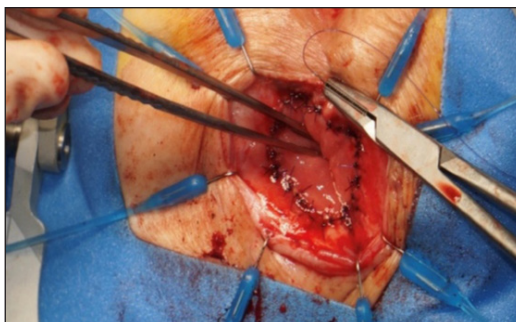


Figure 5. Intact coloanal anastomosis.

Post-Operative Notes:

- Patient had an immediate relief of discomfort related to the rectal prolapse.
- 12-week post-operative visit: almost no further outlet obstruction constipation, improved sphincter function.

TissueRegenix.com
833-567-0767
1808 Universal City Blvd, Universal City, TX 78148

DermaPure® is a registered trademark of Tissue Regenix Wound Care, Inc. dCELL® Technology is a registered trademark of Tissue Regenix Limited. TRX BioSurgery is a Tissue Regenix Group Company. TR053rev02 240906

